

Cleaning Rifton Equipment

Product Cleaning Instructions

General

All of our products are tested to withstand a 10% bleach solution. Because of this, we expect that most commercial disinfectants can be used on our equipment without harming it. However, it is always best to test a small area first. The bleach dilution recommended by the CDC for disinfectant cleaning is around 3%. Clorox wipes should be fine. Alcohol-based cleaners should be fine too.

As a rule of thumb, don't get too much water or cleaning agent into nooks and crannies to prevent corrosion of hardware, etc. To avoid getting the foam inside the padding wet, wipe but do not immerse.

Pacer

As needed, clean with disinfectant wipes or bleach solution. Do not use excessive amounts of water. The straps with hook and loop closures may be laundered. Engage the closures before washing. Do not iron. Wash casters with water after outdoor use. Avoid mud and sand.

Activity Chair

Colored pads can be unsnapped for cleaning, to make it easier to wipe behind and around them.

After each use, clean with disinfectant wipes or bleach solution. Do not use excessive amounts of water. The upholstery should be cleaned in the same manner. You may use a commercial cleaning agent suitable for imitation leather. The harnesses and padded straps should also be wiped down. Do not machine wash. The sandal straps with hook and loop closures may be laundered. Engage the closures before washing. Do not iron.

HTS (Hygiene & Toileting System)

Remove the foam pads for cleaning with soapy water or a mild disinfectant.

After each use, clean with disinfectant wipes or bleach solution. The straps with buckles may be laundered. Engage the buckles before washing. Do not iron. The polyethylene pads are easy to remove to allow cleaning and drying underneath. Wipe pads clean with soapy water or mild disinfectant. HTS pads can be immersed in water (an exception to the general rule above), and smaller pads can be machine washed on a warm setting as needed.

Wave Bath Chair

The Rifton bathing system is designed for easy cleaning. Remove the fabric cover by adjusting both the seat and back to a near vertical position and pulling cover from frame, seat part first. Fabric parts should be washed and dried at a warm (100°F/40°C) setting. Use mild detergent. Rinse well.

The straps with hook and loop closures may be laundered. Engage the closures before washing. Do not iron. Excessive use of bleach and high temperatures may damage the fabric.

Disinfect frame and fabric with a commercial disinfectant or bleach solution. Rinse well.

Tub stand, shower stand, tub transfer base and frame parts may be cleaned using normal cleaning products. Casters will give the best performance if cleaned regularly by spraying from underneath with a high-pressure water hose. Do not use chemicals or lubricants.

Tricycle

As needed, clean Trike with disinfectant wipes or bleach solution. Do not use excessive amounts of water. The straps with hook and loop closures may be laundered. Engage the closures before washing. Do not iron.

TRAM

To minimize risk of infectious disease transmission, clean and disinfect the TRAM between uses by different patients.

The TRAM and its accessories may be cleaned with broad spectrum, multi-purpose disinfectant sprays and wipes or bleach solution. Do not use solutions containing perfumes or staining ingredients. The leg straps, pelvic support and removable back belt (if installed) may be machine washed in cold water and air dried.

IMPORTANT: To avoid damaging the product:

- Do not use excessive amounts of water when cleaning the TRAM.
- Do not use high-pressure spray or steam cleaning.
- Do not clean the piston rod of the TRAM's electric actuator.

E-Pacer

To minimize risk of infectious disease transmission, clean and disinfect the E-Pacer between uses by different patients.

The E-Pacer and its accessories may be cleaned with broad spectrum, multi-purpose disinfectant sprays and wipes or bleach solution. Do not use solutions containing perfumes or staining ingredients. The leg straps, pelvic support and removable back belt (if installed) may be machine washed in cold water and air dried. The straps with hook and loop closures may also be laundered. Engage the closures before washing.

IMPORTANT: To avoid damaging the product:

- Do not use excessive amounts of water when cleaning the E-Pacer.
- Wash casters with water after outdoor use. Avoid mud and sand.
- Do not use high-pressure spray or steam cleaning.
- Do not clean the piston rod of the E-Pacer's electric actuator.

Prone Stander

Clean with disinfectant wipes or bleach solution. The straps, sandals and pads are removable for cleaning.

Supine Stander

Clean with disinfectant wipes or bleach solution. Do not use excessive water on pads and wooden surfaces.

Clean tray with mild soap or detergent. Avoid abrasive cleaners and aromatic spirits. Disinfectants can be used, but no solvents. Never scrape with razor blades or other sharp instruments. Do not clean in hot sun or at elevated temperatures. Do not send through a dishwasher.

Mobile Stander

Clean with disinfectant wipes or bleach solution. Do not use excessive amounts of water. The fleece sleeve can be removed for laundering. Do not iron.

Arm Anchors

As needed, clean with disinfectant wipes or bleach solution. Do not use excessive amounts of water. The upholstery of the elbow anchor and wrist anchor can be removed and cleaned as described, or with a commercial cleaning agent suitable for imitation leather.

Compass Chair

Clean with disinfectant wipes or bleach solution.

Support Station

As needed, clean with disinfectant wipes or bleach solution. Do not use excessive amounts of water. Do not use pine oil cleaners.