

Positioning Checklist for the Rifton HTS

Use this Positioning Checklist as a convenient way to ensure optimal use of the HTS. Write notes to customize your instructions for each individual.

INDIVIDUAL'S NAME

STAFF NAME

DATE



THIS POSITIONING CHECKLIST IS INTENDED FOR CONSISTENT USE OF ADAPTIVE EQUIPMENT FOLLOWING ASSESSMENT BY A HEALTH PROFESSIONAL. THE CHECKLIST DOES NOT REPLACE AN INITIAL CLINICAL EVALUATION FOR CORRECT SIZING, COMPONENT SELECTION, AND APPROPRIATE THERAPEUTIC POSITIONING FOR THE INDIVIDUAL. SOME COMPONENT SIZING IS DEPENDENT ON THE BASE MODEL SIZE.

Base and size

Mobile tilt-in-space with footboard

- small
- medium
- large



Stationary non-tilt with footboard

- small
- medium
- large



Mobile non-tilt with footboard

- small
- medium
- large



Mounting bar for elongated toilet

Mounting bar for round front toilet



Placement for use

- With pan
- Over toilet
- With bowl adapter



On



Over



Off

Seat pad

- Rear-closed seat pad
- Rear-open seat pad



Seat height (from floor)

Specify number of notches visible on chair frame leg _____

Seat depth

Inches (cm) _____

Seat width

Standard hip guides Thick hip guides

Inches (cm) _____

Seat back angle

Forward position (6° anterior; recommended for toileting)

Center position (6° posterior)

Posterior position (18° posterior)



Footrest



Footrest height: Specify number of notches visible on footrest upright bar: _____

Footrest angle

30° forward 20° forward 10° forward 10° back

Calfrest

Ankle straps

Left foot Right foot

Anterior support/tray



Abductor



Deflector

Standard Deflector

Splash guard



Standard



Splash guard

Trunk laterals measure in inches (cm)

Width apart: _____

Height of right lateral (measure from seat to top of support): _____

Height of left lateral (measure from seat to top of support): _____

With chest strap?

Yes No



Laterals



Laterals with chest strap

Butterfly harness



Placement of upper straps prior to securing the buckle

Over the top of the backrest

Through the top slots of the backrest

Through the second slot down from top of backrest

Through the third slot down from top of backrest

Headrest measure in inches (cm)

Height of headrest (measure from seat to top of headrest) _____

